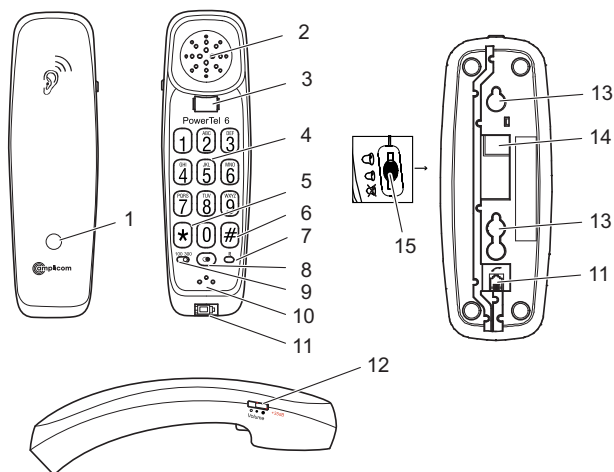
PowerTel 6

Bedienungsanleitung
Operating Instructions
Mode d'emploi
Bedieningshandleiding

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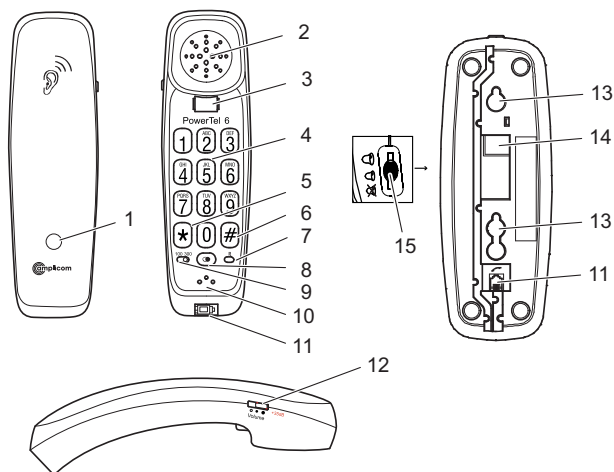
- 1 LED, "phone ringing" /
Amplification +35 dB
- 2 Earpiece
- 3 On-hook contact
- 4 Digit keys
- 5 Star button *
- 6 Hash button
- 7 R-button (Flash)
- 8 Redialling
- 9 Flash time slide control (100 / 300 ms)
- 10 Microphone
- 11 Socket for helix cable
- 12 Receiver volume slide control
- 13 Wall installation bracket
- 14 Telephone connection
- 15 Ringing volume slide control

Connecting the telephone

Connect the helix cable at positions numbered (11). Only use the helix cable supplied because the equipment may not function if another helix cable is used.

Connect the phone (14) to the telephone connection socket using the telephone connection line. Only use the telephone line supplied because the equipment may not function if another telephone line is used

Wall mounting

The telephone can also be installed on a wall. You require two screws in order to fix the telephone on a wall. Drill two holes at a distance corresponding to the slots (100 mm apart) and insert the screws leaving approx. 5 mm projecting from the wall.

Position the telephone on the bracket, fit the screws in the slots and slide the telephone downwards a little.

Taking a call

When the phone rings, lift the receiver and take the call.

Ending a call (on-hook)

Replace the receiver to end the call.

Making a call

Lift the receiver. Wait until you hear the dialling tone. Dial the required telephone number using the digit keys (4).

Redialling

Your telephone stores the last phone number dialled. To redial the last number dialled, lift the receiver and wait until you hear the dialling tone. Press the Redialling button (8). The last number dialled is automatically redialled.

Flash button / Flash time

Press flash key (7) to activate some special network functions.

Flash time (9) - In most situations the flash time of 100 ms should be selected. In some cases, e. g. when connecting the phone to a PABX, it may be necessary to switch to 300 ms. For further information please refer to the user guide of the PABX.

Set the receiver volume

The receiver volume is adjusted using a switch (12).

- Slide the switch to the left:
Low receiver volume.
- Slide the switch to the middle:
Medium receiver volume.
- Slide the switch to the right:
Amplification to +35 dB.

Attention: This phone is capable of providing extremely loud volumes when the amplifier is activated and the volume control is set to maximum. The utmost caution is necessary when the phone is used jointly by people with normal hearing and other with impaired hearing.

GB

Setting the ringing volume

The ringing volume is adjusted using a switch (15).

- : Turn off ringing volume.
- : Low ringing volume.
- : High ringing volume.

Safety notes

Please make sure you read this user guide carefully and keep it for further use.

Intended use

The telephone is suitable for connection within a analogue public telephone network system. Any other use is considered unintended use. Unauthorised modification or reconstruction is not permitted. Under no circumstances open the device or complete any repair work yourself.

Installation location

Make sure your device:

- is on a dry, flat and stable surface.
- isn't a trip hazard with its cables.
- is at least one metre from other electrical appliances to avoid mutual interference.
- isn't exposed to smoke, dust, vibration, chemicals, moisture, heat and direct sunlight.

Disposal

You are obliged to dispose of consumable goods properly in accordance with the applicable legal regulations.

The symbol on this product indicates that electrical and electronic apparatus must be disposed of separately from domestic

waste at suitable collection points provided by the public waste authorities.

Packaging materials must be disposed of according to local regulations.

Helpline

If you need some help and you're in the UK, please call us on 0844 800 6503 (using a UK landline, at the time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays.

If you've a claim on your 2-year guarantee, please contact your supplier.

Maintenance

- Please clean your equipment's surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

Guarantee

AMPLICOM equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life.

The terms of the guarantee do not apply where the cause of the equipment malfunction is the fault of the telephone network operator or any interposed

Helpline UK 0844 800 6503

(See on this page for costs and hours of operation)

private branch extension system. The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies resulting from material of production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property.

Rights to compensation in the case of damage are excluded where there is no evidence or intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return to the sales outlet in which you purchased the AMPLICOM equipment together with the purchase receipt. All rights to claims under the terms of guarantee in accor-

dance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.

Declaration of conformity

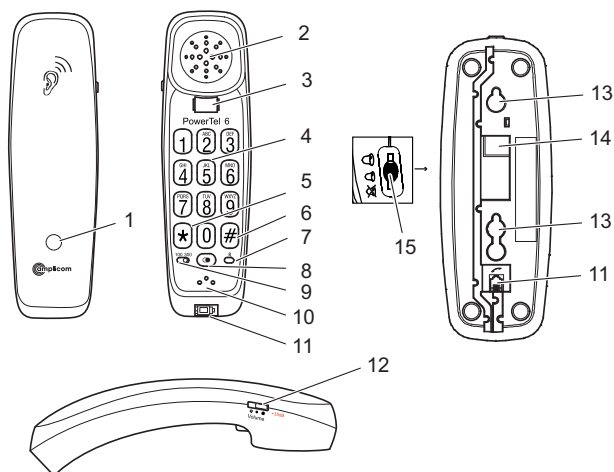
This device fulfils the requirements stipulated in the EU directive:

1999/5 EU directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity.

Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website **www.amplicom.eu**.

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Vertrieb: Audioline GmbH,