



PowerTel 601


User Guide



At a glance

Please open this page for an 'At a glance' guide to your PowerTel 601

Wrist shaker

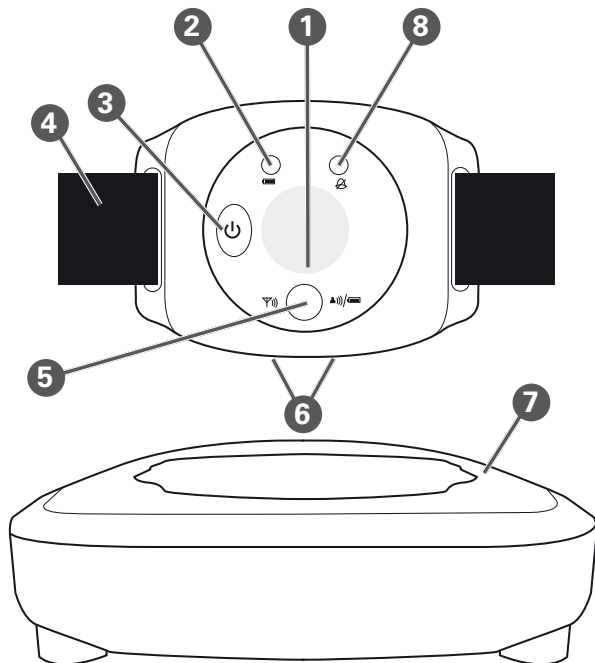

- 1 Emergency button
Note: Can only be used with the PowerTel 680 telephone
- 2 Battery charging red light
- 3 Power on/off
- 4 Wrist strap
- 5 Registration mode/
Out of range/Low battery
2 colour green or red light
- 6 Charging contacts
- 7 Charging unit
- 8 Incoming call green light

Helpline 0844 800 6503 (See page 14 for costs and hours of operation)

Getting started	3
Check box contents	4
Set up your wrist shaker in four simple steps	4
1 Position your charger unit	4
2 Plug in the charger unit.....	5
3 Place the wrist shaker in the charger unit	5
4 Register your wrist shaker to your PowerTel telephone	6
Registration key guide.....	8
Using your wrist shaker	9
Switch on.....	10
Switch off	11
Wrist strap	11
Make sure your wrist shaker is in range.....	11

Battery low	12
Incoming calls	12
Help and support	13
Troubleshooting guide	14
General information	17
Safety information.....	18
Charger unit location.....	18
Power supply	18
Energy efficient power adapter	19
Power failure	19
Medical equipment.....	19
Disposal	20
Maintenance.....	20
Guarantee.....	21
Declaration of conformity	23

Getting started

Getting started

Important: please make sure you've read the safety information on page 18 before you set up your wrist shaker.

Check box contents

You should have:

- 1 wrist shaker with a wrist strap attached
- 1 charger unit with power adapter plug attached

Set up your wrist shaker in four simple steps

1 Position your charger unit

Make sure your charger unit:

- is on a dry, flat and stable surface
- isn't a trip hazard with its power cable
- is at least one metre from other electrical appliances

2 Plug in the charger unit


Plug your charger unit into your power supply. And switch on at socket.

Important: use only the power adapter supplied with your PowerTel 601 otherwise it may not work and it could damage the product.

3 Place the wrist shaker in the charger unit

When you place your wrist shaker on its charger unit, make sure its two connection points are facing down into the unit. You'll know your wrist shaker is charging properly when you see a red light on it.

Mains power
wall socket

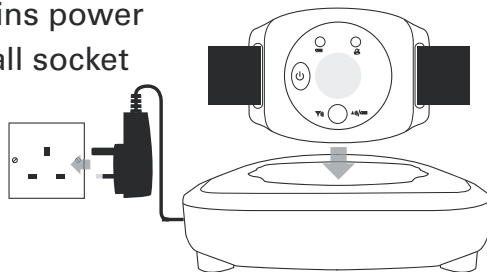



Your wrist shaker has an internal rechargeable battery which must be charged before you can use it. So it can charge fully, your wrist shaker needs to charge for at least 10 hours (make sure the power socket is switched on!).

Note: when fully charged, your wrist shaker's battery should last five days. The larger red light will flash slowly when the battery is low and you will need to charge it for about another ten hours.

4 Register your wrist shaker to your PowerTel telephone

Important: if you got your wrist shaker separate to your phone, you need to register it for it to work.


- i. Turn on the wrist shaker by pressing and holding the  button for about five seconds.
- ii. Press and hold the  button again for about 12 seconds, until the registration light flashes green quickly. The wrist shaker will be in registration for about 1 minute.

iii. Activate registration mode on your PowerTel phone.

If you use a PowerTel 500, 502 or PowerTel 580 phone:

Press and hold the registration key  for about ten seconds until the charging light flashes.

If you use a PowerTel 580 combo:


Press and hold  until you see **Register** on the display.

Note: if you can't see the registration key , see page 8.


iv. You'll know your wrist shaker is registered and ready to use when the registration light is a steady green (i.e. it isn't flashing).

Note: the registration light will flash green and red if your wrist shaker either isn't registered or is out of range. Move closer to your PowerTel telephone to ensure you are within range.


Registration key guide

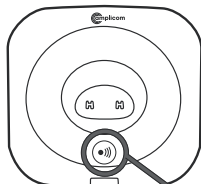

PowerTel
500/502

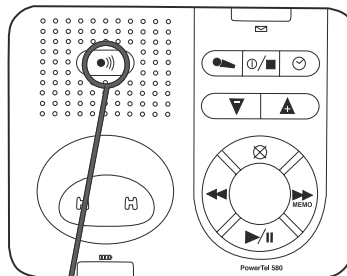

PowerTel 580

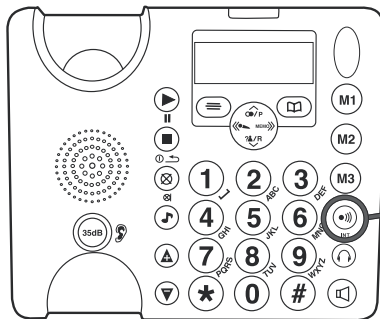


PowerTel 580 Combo


Press and hold
registration key
for at least
10 seconds

Using your wrist shaker

Using your wrist shaker


Switch on

Press and hold the  button for about five seconds. You'll see the green light with the warning triangle come on.

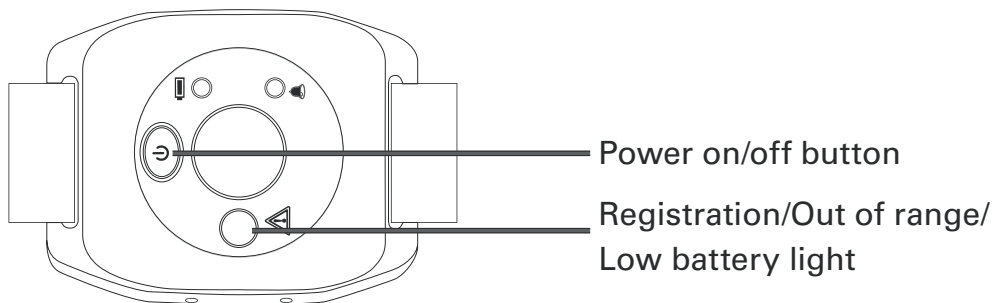

Note: if the light flashes green then red it means your wrist shaker either isn't registered or is out of range. Move closer to your PowerTel telephone to ensure you are within range. If the light still flashes, see page 6 for registration instructions.

Switch off

Press and hold the  button for about five seconds. The green light will go off.

Wrist strap

The wrist strap comes pre-attached to the wrist shaker but if you need to re-attach or adjust it follow these instructions.

Thread the wrist strap through the slot on one side of the wrist shaker (front to back), around the back and then through the other slot (back to front), just like you would with a wrist watch.

You can now wear your wrist shaker and tighten comfortably with the velcro fastening.

Make sure your wrist shaker is in range

If your wrist shaker is not in range, it won't work. You can tell it's out of range when the large light flashes green and red. If this happens, you need to move closer to your PowerTel phone for your wrist shaker to work.

Battery low

If the battery is low, the large light will turn red and will flash. You need to place the wrist shaker in its charger base for 10 hours to charge the battery.

Incoming calls

You know you've got an incoming call when your wrist shaker vibrates and a bell symbol flashes green on its display.

Answer and end your call in the usual way using your PowerTel phone.

Help and support

Help and support

If you need some help and you're in the UK, please first check and see if your problem can be solved using the troubleshooting guide below. If it can't, please call us on 0844 800 6503 (using a UK landline, at time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays. If you've a claim on your 2-year guarantee, please contact your supplier.

Troubleshooting guide

My wrist shaker doesn't vibrate when I've an incoming call

Make sure you've registered your wrist shaker. If you got your wrist shaker separate to your phone, you need to register it. Please see Step 4 of Getting Started on page 6.

If you have registered your wrist shaker and it still doesn't vibrate with an incoming call, you may have moved out of range of your phone. Move closer to your phone.

My display is flashing red

The battery is low. Recharge your wrist shaker by placing it on your charger unit. To fully charge it will take about ten hours.

General information

Safety information

Please make sure you read this User Guide carefully.

Under no circumstances should you open the product or complete any repair work yourself.

Charger unit location

Make sure your charger unit:

- is on a dry, flat and stable surface
- isn't a trip hazard with its power cable
- is at least one metre from other electrical appliances
- isn't exposed to smoke, dust, vibration, chemicals, moisture, heat and direct sunlight.

Power supply

Important: use only the power adapter supplied with your PowerTel 601 otherwise it may not work and it could damage the product.

Energy efficient power adapter

The included mains adapter meets the eco-design requirements of the European Union (Directive 2005/32/EG). Thus, its power consumption is considerably lower, both in operated as well as in non-operated mode, compared with the mains adapter of previous design.



- Energiesparend
- Economisant l'énergie
- A risparmio di energia
- Energy efficient

Power failure

This product cannot be used in the event of a power failure. Please make sure you've alternative arrangements in place for emergency calls.

Medical equipment


Do not use your wrist shaker:

- near emergency or intensive care medical equipment
- during an electrical storm

It is recommended that if you have a pacemaker fitted you check with a medical expert or your health adviser before using this product

Disposal

You are obliged to dispose of consumable goods properly in accordance with the applicable legal regulations.

The  symbol on this product indicates that electrical and electronic apparatus and batteries must be disposed of **separately** from domestic waste at suitable collection points provided by the public waste authorities. Batteries can also often be disposed of at the point of sale. Packaging materials must be disposed of according to local regulations.

Maintenance

- Please clean your equipment's surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

Guarantee

AMPLICOM equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life. The terms of the guarantee do not apply where the cause of the equipment malfunction is the fault of the telephone network operator or any interposed private branch extension system. The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies resulting from material or production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property.

Rights to compensation in the case of damage are excluded where there is no evidence or intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return to the sales outlet in which you purchased the AMPLICOM equipment together with the purchase receipt. All rights to claims under the terms of guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.



Declaration of conformity

This device fulfils the requirements stipulated in the EU directive:

1999/5 EU directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity.

Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website **www.amplicom.eu**



Distributed by: Audioline GmbH, D-41460 Neuss

FEB 10 – ISSUE 1.