NL 100

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Mode d'emploi

Operating Instructions

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NL 100

D

F

GB

NL

NL 100

Bedienungsanleitung	3
Mode d'emploi	8
Operating Instructions	13
Gebruiksaanwijzing	18

Sicherheitsinformationen

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AUDIOLINE GmbH
Neuss
Internet: www.amplicom.eu
E-mail: info@audioline.de

Safety Information

Please read this operating instruction manual thoroughly.
Under no circumstances open the device or complete any repair work yourself.

Hearing aid compatibility

This product is compatible with most popular hearing aids on the market. However, due to the wide range of hearing aids available, there is no guarantee that the product will function 'problem free' with every model

Disposal

You are obliged to dispose of consumable goods properly in accordance with the applicable legal regulations. The adjacent symbol on the product indicates that electrical and electronic apparatus no longer required must be disposed of separately from domestic waste.

Electrical and electronic devices must be disposed of at suitable collection points provided by the public waste authorities.

Packaging materials must be disposed of according to local regulations.

Starting Up

Starting Up

1. Check the contents of the package

Contents	NL100
Neck Loop NL100	1
AAA Battery	1
2.5 mm to 3.5 mm Product Adaptor	1
Operating Manual	1

2. Insert the battery

Insert the 1 x AAA battery into the battery compartment on the back of the NL100. Take care to note the '+' and '-' markings and position the battery the correct way around. Close the battery compartment cover.

Caution: Do not use rechargeable batteries

3. Connect the NL100

Connect the NL100 to the headset socket of your PowerTel handset as shown in the diagram and put the neck loop over your head.

Using the NL 100

The NL 100 uses an induction loop system which will work together with your hearing aid, when set to the T position, to help you hear sounds more clearly.

1. Slide the ON/OFF switch to 'ON'. The green power LED will light up.

If the power LED turns red, the battery power is low and you should change the battery as soon as possible.

2. Switch your hearing aid to 'T' or 'MT' mode.
3. Adjust the volume using the wheel on the NL 100.
4. Your NL 100 is now ready for use and you can speak to your callers using the built in microphone.
5. Slide the ON/OFF switch to 'OFF' at the end of your call to save battery power. The power LED will go out.

You can use the product adaptor supplied with your NL 100 to connect to compatible MP3 devices or mobile phones. Follow the steps above.

Maintenance/Guarantee

Maintenance

- Clean the product surfaces with a soft, fluff-free cloth
- Never use cleaning agents or solvents

Guarantee

AMPLICOM equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble-free functioning and a long service life. The terms of the guarantee do not apply where the cause of the equipment malfunction is the fault of the telephone network operator or any interposed private branch extension system. The terms of guarantee do not apply to the rechargeable batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies resulting from material or production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee. In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property.

Rights to compensation in the case of damage are excluded where there is no evidence or intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return it to the sales outlet in which you purchased the AMPLICOM equipment together with the purchase receipt. All rights to claims under the terms of the guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase and transfer of our products, claims under the terms of guarantee can no longer be asserted.

Declaration of Conformity

This device fulfils the requirements stipulated in the EU directive: 1999/5 EU directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity.

Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website www.amplicom.de.

Distributed by: Audioline GmbH, D-41460 Neuss

Important Notes on the Guarantee

The period of guarantee for the AMPLICOM product begins on the date of purchase. Rights to claims under the terms of guarantee relate to deficiencies resulting from material or production faults. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, normal wear and tear, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

CLAIMS UNDER THE TERMS OF GUARANTEE
MAY ONLY BE ASSERTED
TO THE RETAILER WHERE THE EQUIPMENT WAS
PURCHASED,
WHEREBY THE PURCHASE RECEIPT MUST ALSO
BE SUBMITTED.

TO PREVENT UNNECESSARY LONG DELAYS,
PLEASE DO NOT SEND
EQUIPMENT UNDER COMPLAINT TO OUR
OFFICE ADDRESS.
IN THE CASE OF QUESTIONS CONCERNING
OPERATION OR THE
TERMS OF GUARANTEE, PLEASE REFER TO THE
OPERATING MANUAL
OR CONTACT OUR HOTLINE:
0844 800 6503

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Verkoop: Audioline GmbH, 41460 Neuss (D)